


1-Second Dental Curing Light Instructions for Use

One Second LED Curing Light

Environmental white plastic shell; 10W high power LED; strong, one second, gradually strong 3 modes; large capacity lithium battery; shut down automatically without any operation in 40 seconds; low battery warning; suitable for all brands of resin materials; long standby time of 90 days or more; be charged one time, in accordance with each working time of 20 seconds, it can be used continuously for more than 200 times.

Operating mode diagram

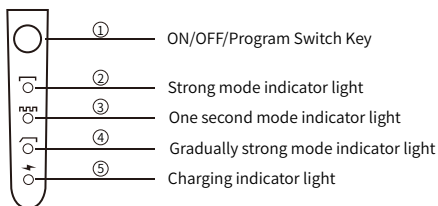

Operating Details

① Power On/Off

Press Key ① for starting/shutting down the machine.

② Program Switch

Long press Key ①, when hear the voice "di" release hand, at the same time indicator light flashing, then repress Key ① to change different working modes. After choosing appropriate working mode, it will be settled automatically without any operation in 3 seconds. Then press Key ① again, the machine start to work.

③ strong working indication: irradiation with strong intensity

④ one second working indication: irradiation with flashing

⑤ gradually strong working indication: irradiation with intensity from low to high

⑥ charging indication: the light will be flashing when machine is in charging

AT&M Biomaterials Co., Ltd.

Building 1, Zhongguancun Science and Technology Park at Pinggu, Courtyard No. 46,
Pingwang Street, Pinggu District, Beijing, China

Tel:+86-10-69778208

Email: info@atmbio.com

Website: www.atmbio.com