

Compules Delivery Device Instructions for Use



Compules Delivery Device, this applicator is used to precisely deliver resin materials into cavity preparations, reducing material waste and the risk of cross-contamination.

Operating Steps

1. Loading the Resin Compule
 - Open the applicator handle to its widest position to fully retract the plunger.
 - Insert the resin compule into the notched opening of the gun, collar end first. Ensure the collar is fully seated in the slot.
 - After confirming the compule is secure, remove the colored protective cap from the front end.
2. Dispensing the Material
 - Position the applicator tip at the cavity preparation site. The tip can be rotated 360° for optimal angle adjustment.
 - Apply slow, steady pressure to the handle to dispense the resin material.
 - Do not apply excessive force, as this may cause material splatter or damage to the applicator tip.
 - Once dispensing is complete, release the handle. The plunger will automatically reset.
3. Removing the Resin Compule
 - Fully open the applicator handle to retract the plunger completely.
 - Gently press down on the front end of the resin compule to remove it from the applicator.

Safety and Operating Precautions

- For professional use only: Must be operated by licensed dentists or dental technicians.
- Do not apply excessive force: Apply even pressure to avoid breakage of the applicator tip or material overflow.

Storage and Maintenance

- The applicator body is reusable. After use, disinfect and sterilize according to the instructions (refer to your institution's standard protocols for dental instruments).
- Do not store in high temperature, humid, or corrosive environments.



AT&M Biomaterials Co., Ltd.

Building 1, Zhongguancun Science and Technology Park at Pinggu, Courtyard No. 46,
Pingwang Street, Pinggu District, Beijing, China

Tel:+86-10-69778208

Email: info@atmbio.com

Website: www.atmbio.com