

Attention

This product is intended for personal care use only and cannot be used as a medical device. The wifi connection provided by the AI dental camera is not connected to the Internet.

The camera on this device is with marco lens, and it can not be used as a monitoring device.

Children are prohibited from using this product on its own, Children must always use it under adult supervision. Please keep the product in a cool and dry place, keep it away from fire and corrosive substances.

Please keep the product out of the reach of children.

Please do not clean this product by water, use a dust-free cloth to gently wipe the product lens.

It is forbidden to use when the device is charging.

At the end of the service life, please correctly dispose this product according to the local environmental policy.

Warranty

1. Definition of coverage period: within six months from the date of purchase, it is a basic coverage period, and the user does not need to pay additional fees, and the user can also choose to pay to extend the duration of the coverage period.

2. Within the six months basic guarantee period, if it occurs non-man-made damage performance failure, after confirmed by after-sales service center , with the invoice users will be able to get free repair service.

Free Service Will Not Be Available In The Following Cases:

1. Man-made damage caused by improper use, such as to use an improper power supply, to use of improper accessories, improper installation, to use of non-original accessories, to use the AI dental camera not in accordance with the instructions, misuse or damage caused by negligence.

2. Damage caused by force majeure (e.g. natural disasters, etc.).

3. Private dismantling or repair by persons not authorized by the company.


4. Product aging, wear and tear caused by normal use, but not affecting the normal use of the product.

5. Unable to provide proof of purchase or record of purchase.

Product Parameters

Product size: 220×28×24mm	Product net weight: 35g
Lens pixels: 2 million	Effective focal length: 20~35mm
Resolution: 1280x720	Battery capacity: 500mAh
Input voltage: DC 5V	Anti Shake: Support
Support system version: Android 4.4 or IOS 5.3 or above	

Operating

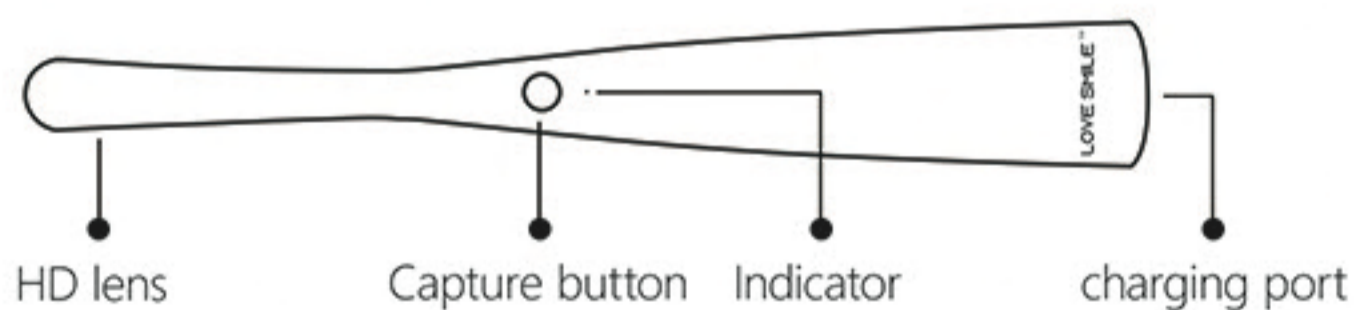

Power On The Device: Long press the capture button for 2 seconds, the indicator light is flashing blue and the lens light is on at the same time that is the power on state. Before using it, please make sure that the battery is sufficient. In case the device can not be turned on, please try to charge the device.

Software Download: Scan the code below to download and install Uteeth APP.

Download App From Below QR Code:




(iOS)



Uteeth (Android)

LOVE SMILE™

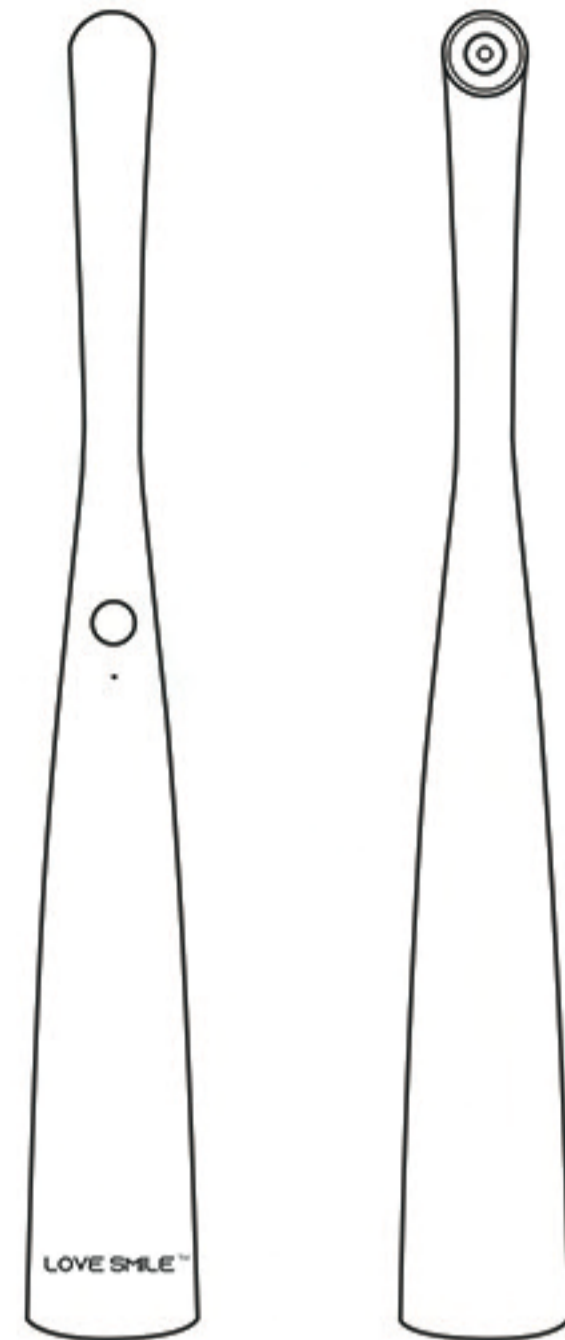

AI DENTAL MIRROR C6 PRODUCT MANUAL

Device Connection: Turn on the device. Take out the smart phone and go to wifi setting page, search wifi with name 'Acreate_XXXXX' and connect to this wifi, password 12345678. Then run Uteeth App on the phone, click 'immediate inspection' or 'AI inspection'. It means successful connection when real-time image shows up on phone screen.

Oral Examination: Hold the device, locate the lens at proper position of the teeth to be examined, short press the capture button to take a picture. When photo is frozen on phone screen, it will automatically identify pathological symptoms and marks them with different colors. Press the capture button to return to preview state.

Oral Report: After taking the photos, return to the APP home page and click 'AI Assessment Report' to view the Report.

Indicator Light Description: When blue light is always on, it means wifi connection is successful. when blue light is slow flashing, it means no wifi connection, When red light is slow flashing, it means the device is under charging, when red light is always on, it means the battery is fully charged.

Common Problems & Solutions

Blurred Image:

1. Gently wipe the lens with a dust-free cloth or alcohol pad.
2. Make sure the lens to the teeth is at proper distance, the suggested focal distance is 20-35mm

Screen Lag:

1. The phone is too far away from the camera. Get the camera closer to smart phone.
2. The device has very low power, which makes the wifi signal very weak. Please charge the device and then use it.

Cannot display video streaming:

1. If the APP cannot display video streaming normally after connecting to the device's hotspot network, please turn off the mobile phone data network and try again.

Get prompts: "Cannot access the Internet" or "The network quality is poor, do you allow the network assistant to switch to a better network?"

1. Please choose to stay connected to the current device hotspot network.

Don't show reports:

1. Please make sure your mobile network is connected back to the Internet.

WIFI signal cannot be transmitted:

1. the lower the battery, the weaker the signal will be, so please keep the battery sufficient. If there are too many WIFI signals in the environment, which will affect the signal of this product, please retest in an area with less WIFI signals.

When the above steps fail to solve the problem, you can charge and reboot the dental camera, i.e., charge the product for one minute and retest.